


BL

BONE LEVEL implant

C-TECH INSTRUMENTS NECESSARY FOR BL IMPLANT SURGERY

DENTAL IMPLANT

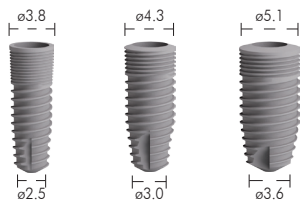

6 mm		BL-4306*	BL-5106*
7 mm	BL-3507	BL-4307	BL-5107
9 mm	BL-3509	BL-4309	BL-5109
11 mm	BL-3511	BL-4311	BL-5111
13 mm	BL-3513	BL-4313	BL-5113
15 mm	BL-3515	BL-4315	BL-5115



Locator drill
CT-2020



2.0 external irrigation drill (Ø 2.0)
CT-1720E



3.5 external irrigation drill (Ø 3.0)
CT-1735E



4.3 external irrigation drill (Ø 3.6)
CT-1743E



5.1 external irrigation drill (Ø 4.6)
CT-1751E



Counterbore
CT-2035E



3.5 Bone Tap
CT-7154E



4.3 Bone Tap
CT-7155E



5.1 Bone Tap
CT-7151E



Drill Extender
CT-2000



Manual Hex Driver
CT-9025S



1.25mm Hextool Torque Wrench
Attachments CT-8051



1.25mm Hextool Torque Wrench
Attachments (Long) CT-8052



Implant Latch Driver
CT-E9040



Implant Ratchet Driver (Short)
CT-E7001



Implant Ratchet Driver (long)
CT-E7001L



Torque Wrench (50 Ncm)
CT-8010



Depth Gauge (3.5 mm)
CT-E9007



Paralleling Pins, qty. 2
(1.6 mm & 2.0 mm) CT-9000



Set metal stopper (L.9/11/13/15)
CT-Stop 07/08/09/10



Implant Latch Driver
BL-E9040



Implant Ratchet Driver Short
BL-E7001



Implant Ratchet Driver Long
BL-E7001L



Abutment Extractor Screw
BL-6060

C-TECH
IMPLANT

www.c-tech-implant.com

BL BONE LEVEL implant SURGICAL PROCEDURE



SURGICAL PROCEDURE BL Ø 3.5 IMPLANT

D2/D3

D1 ADDITIONAL STEPS

Explanation of drill marking



CT-2020
Locator Drill

800 - 1000 rpm
Depth: Length 4 mm

5 mm



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)

Depth: Length of implant
100 - 750 rpm




CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)

Depth: Length of implant
100 - 750 rpm



BL-E7001L
Implant
ratchet driver

Insertion Speed: 0 - 50 rpm
Insertion Torque: 35 - 50 ncm

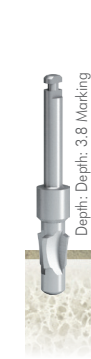


CT-2035E
Irrigated
Counterbore

Depth: 3.8 Marking
on Counterbore 100 - 500 rpm




CT-7154E
3.5 Bone Tap
(Outer Ø
3.4mm for use
in hard bone)

Depth: Length of implant
10 - 15 rpm

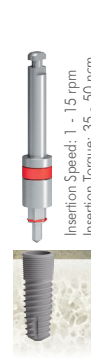

BL-E9040
Implant latch

Insertion Speed: 1 - 15 rpm
Insertion Torque: 35 - 50 ncm

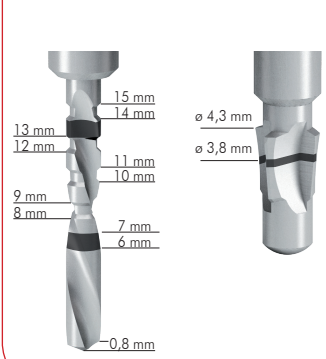

SURGICAL PROCEDURE BL Ø 4.3 IMPLANT

D2/D3

D1 ADDITIONAL STEPS



CT-2020
Locator Drill

800 - 1000 rpm
Depth: Length 4 mm

5 mm



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)

Depth: Length of implant
100 - 750 rpm



CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)

Depth: Length of implant
100 - 750 rpm



CT-1735E
4.3mm Irr. Drill
(Outer
Ø 3.9mm)

Depth: Length of implant
100 - 750 rpm



BL-E7001L
Implant
ratchet driver

Insertion Speed: 0 - 50 rpm
Insertion torque: 35 - 50 ncm




CT-2035E
Irrigated
Counterbore

Depth: Length of implant
100 - 500 rpm



CT-7155E
4.3 Bone Tap
(Outer Ø
4.2mm for use
in hard bone)

Depth: Length of implant
10 - 15 rpm

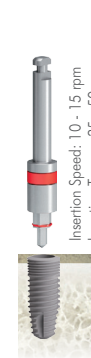

BL-E9040
Implant latch

Insertion Speed: 10 - 15 rpm
Insertion Torque: 35 - 50 ncm



SURGICAL PROCEDURE BL Ø 5.1 IMPLANT

D2/D3

D1 ADDITIONAL STEPS



CT-2020
Locator Drill

800 - 1000 rpm
Depth: Length 4 mm

5 mm



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)

Depth: Length of implant
100 - 750 rpm



CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)

Depth: Length of implant
100 - 750 rpm



CT-1735E
4.3mm Drill
(Outer
Ø 3.9mm)

Depth: Length of implant
100 - 750 rpm



CT-1735E
5.1mm Drill
(Outer
Ø 4.6mm)

Depth: Length of implant
100 - 750 rpm




BL-E7001L
Implant
ratchet driver

Insertion Speed: 0 - 50 rpm
Insertion Torque: 35 - 50 ncm



CT-7151E
5.1 Bone Tap
(Outer Ø
5.0mm for use
in hard bone)

Depth: Length of implant
10 - 15 rpm

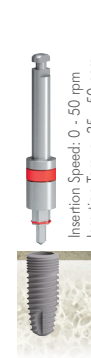

BL-E9040
Implant latch

Insertion Speed: 0 - 50 rpm
Insertion Torque: 35 - 50 ncm